

PRO-TEC II™ SYRINGE SHIELD

OPERATION MANUAL

007-700
007-800
007-900
007-909
007-835
007-837
007-903
007-915
007-916



BIODEX

Biodex Medical Systems, Inc.

20 Ramsay Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (In NY and Int'l. call 631-924-9000).

Fax: 631-924-9241 Email: sales@biodex.com, www.biodex.com

PRO-TEC II™ SYRINGE SHIELD

This manual contains operation procedures for the following Biodex products:

#007-700	Syringe Shield, 1 cc
#007-800	Syringe Shield, 3 cc
#007-900	Syringe Shield, 5 cc
#007-909	Syringe Shield, 10 cc
#007-835	Syringe Shield, 2 cc and 2.5 cc for Omni Syringe
#007-837	Syringe Shield, 2 cc and 2.5 cc for Once Syringe
#007-903	Syringe Shield, 5 cc for Once Syringe
#007-915	Syringe Shield, 5 cc for Terumo Syringe
#007-916	Syringe Shield, 10 cc for Terumo Syringe

INTRODUCTION

The Pro-Tec II Syringe Shield is constructed of virtually indestructible tungsten. With proper care, it will last indefinitely.

To use, merely insert syringe through the tubular body, align syringe volume calibrations to the lead glass window, and tighten screw gently until firm pressure is exerted against the syringe body. Syringe may now be safely used. To withdraw syringe, release screw pressure and extract syringe.

NOTE:

- A. For your convenience, Model 007-900 (5cc) shield includes a special adapter insert to accommodate different brands of syringes.*
- B. Pressure screw is made of brass. As brass is softer than the tungsten shield body, the screw will eventually strip. We have enclosed extra screws for your convenience. If you require additional screws at any time, please call or write. We will supply them at no charge.*
- C. The lead glass window has been attached to the shield with a spring clip. Should the glass be damaged or destroyed in use, it may be replaced by removing the old glass and sliding a new piece of glass into place in the shield.*

MAINTENANCE

As needed, clean with alcohol. Do not clean lead glass with decontaminating solution as this may fog the glass. Do not soak in water.


REPLACEMENT GLASS

127-700 Syringe Shield Replacement Glass, 1 cc
For 007-700

127-789 Syringe Shield Replacement Glass, 3 cc and 5 cc
For 007-800, -900, -835, -837, -903, -915

127-999 Syringe Shield Replacement Glass, 10 cc
For 007-909, -916

Authorized European Community Representative: Prothia, Paris, France

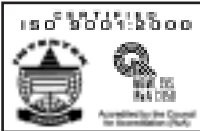

BIODEX

Biodex Medical Systems, Inc.

20 Ramsay Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (In NY and Int'l. call 631-924-9000),

Fax: 631-924-9241 Email: sales@biodex.com, www.biodex.com