

PRO-TEC III SYRINGE SHIELD

OPERATION MANUAL

007-734
007-735
007-736
007-738
007-745
007-746
007-748
007-755



BIODEX

Biodex Medical Systems, Inc.

20 Ramsay Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (In NY and Int'l. call 631-924-9000).

Fax: 631-924-9241 Email: sales@biodex.com, www.biodex.com

PRO-TEC III™ SYRINGE SHIELD

This manual contains operation procedures for the following products:

007-734	Syringe Shield, 1 cc
007-735	Syringe Shield, 3 cc
007-736	Syringe Shield, 5 cc
007-738	Syringe Shield, 10 cc
007-745	Syringe Shield, 3 cc for BD Plastipak and Terumo 3 ml syringes
007-746	Syringe Shield, 5 cc for Once syringes
007-748	Syringe Shield, 10 cc, Euro
007-755	Syringe Shield, 2.5 cc for Terumo syringes

The Pro-Tec III Syringe Shield features a unique Safe-T-Lock design that automatically grips and secures the syringe in place and allows the syringe to be released — without additional handling — at the simple press of a button. Constructed of a virtually indestructible tungsten alloy, the Pro-Tec III offers a 2.0 mm lead equivalency and will reduce exposure from Tc-99m by greater than 94%. The lead glass viewing panel provides maximum protection and visibility, enhanced by an interior white coating to ensure easy reading of the syringe and its contents.

Operation

Insert the syringe through the tubular shield and align the syringe volume calibrations to the lead glass window. Once the syringe is fully inserted into the shield, the shield lever lock automatically engages to hold it securely in place.

To remove the syringe from the tubular shield, invert the syringe shield over a shielded Sharps container and simply press and hold down the lever. With the lever depressed, the syringe will drop right out of the shield without additional handling.

NOTE: For your convenience, a special adapter to accommodate syringes with longer barrels is available free of charge for use with the Model 007-735 (3 cc) Syringe Shield. Contact Biodex if you need this adapter (Model 007-739, Spacer, Pro-Tec III) To use the adapter, simply slide it over the syringe before inserting the syringe into the shield. (Be sure not to accidentally dispose of the adapter when removing the syringe.)



Safe-T-Lock design quickly engages or releases syringe.

Maintenance

As required, clean the syringe shield with alcohol. Do not clean lead glass with decontaminating solution as this may fog the glass. Do not soak in water.



WARNING: Do not autoclave, steam-clean or use decontaminating solution on this device.



ATTENTION: Ne pas stériliser par chaleur sèche ou humide. Ne pas utiliser de liquide de décontamination.

Replacement

127-735 Glass, Replacement for models
007-735, 007-736, 007-745, and 007-746

127-738 Glass, Replacement for models
007-734, 007-738, 007-748

Authorized European Community Representative: Prothia, Paris, France



BIODEX

Biodex Medical Systems, Inc.

20 Ramsay Road, Shirley, New York, 11967-4704

Tel: 800-224-6339 (*In NY and Int'l, call 631-924-9000*).

Fax: 631-924-9241 Email: sales@biodex.com, www.biodex.com